



MESI

Simplifying Diagnostics

Quick Guide


MESI mTABLET

Wireless Medical Tablet System


WHAT IS IN THE BOX?

Medical grade tablet unit


Model
MTABMD

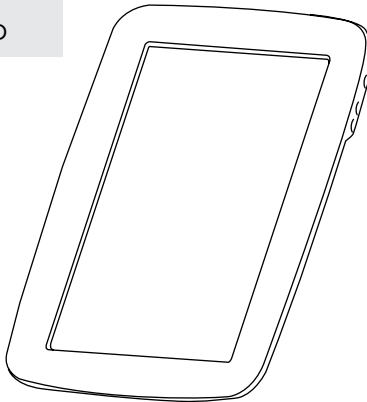

Shielded Ethernet cable - 2m

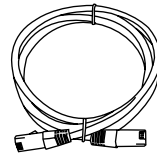


AC/DC Power supply for
MTABDSW.

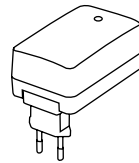

MESI mTABLET docking station with access point.


Model
MTABDSW

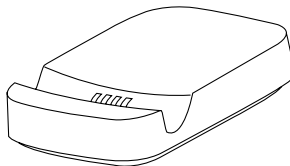

Before using the device for the first time, read the user manual carefully and follow the recommendations here: www.mesimedical.com/support/mTABLET/IFU.

FIRST USE

1. Connecting the dock

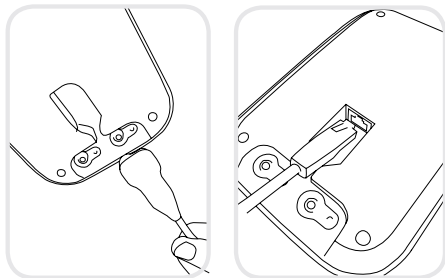

Connect your charging station to the mains and to the local area network with the provided network cable and power adapter.

2. MESI mTABLET set up



Before first use, set MESI mTABLET on the charging dock to fully charge it.

When MESI mTABLET is charged, press the ON/OFF button on the right side of the tablet to turn on the device.

3. First boot



Since the MESI mTABLET offers several languages, select the **language** you wish to use from the drop-down menu. This setting can be changed later in the user settings.

The charging dock for the MESI mTABLET comes with **a built-in Wi-Fi access point** to which the MESI mTABLET will connect automatically on first use.

If another wireless network is desired, please select it from the list and enter the security information.

Wi-Fi is not required for the essential functions of the MESI mTABLET, but it is necessary for measurements to be synced to the MESI mRECORDS web application.

PAIRING DIAGNOSTIC MODULE

Before any recordings can be performed, the appropriate module needs to be paired to your MESI mTABLET. Please take the MESI mTABLET and open the user tab.




Step 1

Go to **User profile > Settings > Modules > +Add module**


Step 2

Press the button on top of the appropriate module.

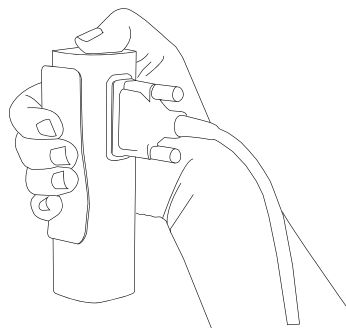

Step 3

When the MESI mTABLET establishes a connection with the module, the light on top of the module will change. Confirm the pairing process by pressing the button on top of the module again.

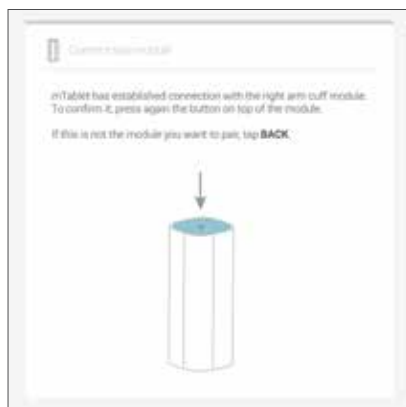

Step 4

The final screen indicates successful connection between the MESI mTABLET and the wireless module. You can access all technical information about the connected modules in **User profile > Settings > Modules**.



WARNINGS



MESI mTABLET users must be adequately trained to use the device. Before first use of the device, users must carefully read the entire usage instructions and follow the instructions for use of the connected equipment.



Use only accessories and other parts recommended or supplied by MESI. Use of other than recommended or supplied parts may result in injury, inaccurate information and/ or damage to the unit.



Do not use this unit in areas where there is any danger of explosion or the presence of flammable gases such as anaesthetic agents.



The AC/DC power supply must be connected to an easily accessible socket (the AC/ DC power supply also serves as galvanic isolation). To prevent electric shock, do not disassemble the unit. There are no serviceable parts inside. Refer servicing to qualified personnel only.



Only use non-aggressive cleaning agents to clean the device. The device may be wiped with a damp cloth.



Protect the device from moisture and liquids and extremely high/low temperatures. Also, protect the device from mechanical stress and do not expose it to direct sunlight, as this may cause the device to not function properly.



Never carry out repairs of any kind yourself. Do not open the device. The device does not contain any parts which can be replaced by the user. Do not alter or adapt the device. If a defect occurs, consult your dealer or distributor.

DISTRIBUTOR INFORMATION

MESI CONTACT INFORMATION

Address MESI, development of medical devices, Ltd
Leskoškova cesta 11a
SI-1000 Ljubljana
Slovenia, European Union

Telephone +386 (0)1 620 34 87
E-mail info@mesimedical.com
Website www.mesimedical.com

USER ACCOUNTS

1. Register your user account

The screenshot shows the 'Register' screen in the MESI app. It features a 'Login' button for existing users and a 'Register' button for new users. The registration form includes fields for Email, Password, Confirm password, Country, Telephone, and Role (Doctor or Nurse). A 'Terms and agreement' section is at the bottom with a green 'Register' button.

The screenshot shows the 'Profile' screen in the MESI app. It features fields for Name, Surname, and Specialization. Below these fields is a 'Profile picture' section with a 3x3 grid of avatars. At the bottom are 'CANCEL' and 'OK' buttons.

Each user accessing the MESI mTABLET needs to register an account, which will also enable them to access data securely on other devices (personal computer, tablet or phone if needed).

2. Working groups

The screenshot shows the 'Working group' screen in the MESI app. It features a confirmation message: 'We have created a new working group Carlos Blanco'. Below this are fields for Description, Status, Practice name, Practice phone number, Practice fax number, Practice address, Practice email, and Practice website. At the bottom are 'CANCEL' and 'OK' buttons.

Each doctor can be the owner of only one working group, but can be invited into multiple working groups. At the same time, they can add multiple nurses or doctors into their working group. Nurses, on the other hand, can be part of many working groups, but they can't create one or add more people to it. This system enables greater oversight of patient data access.

MESI mRECORDS

MESI mRECORDS can be accessed with the same user login data as MESI mTABLET at <https://mrecords.mesimedical.com>. Since mRECORDS is a web app this gives you access from anywhere and anytime. All measurements and patient data is synced between the MESI mTABLET and MESI mRECORDS, thus giving the medical expert access to recorded measurement instantly.

For more information and tutorials please log-in or register your account on <https://mrecords.mesimedical.com>.

